


AndraSnare[®]/Loopmaster[®] - Sochman Snare

3D nitinol snares for foreign body retrieval

AndraSnare® & AndraSnare® Micro

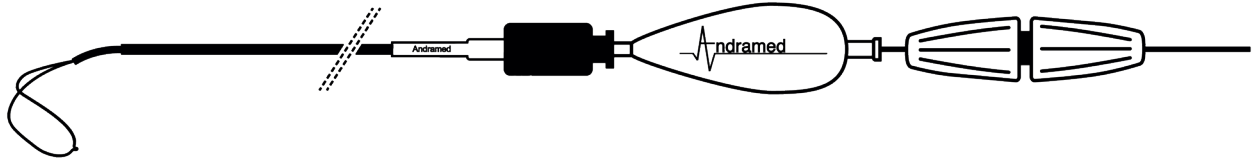

- Nitinol snare for the interventional retrieval of intravascular foreign bodies in the cardiovascular system
- Flexible nitinol loop that re-assumes its original shape after exiting the catheter
- High x-ray visibility through tungsten-gold-coating of the loop
- Radiopaque shaft material plus additional x-ray marker

Order Codes AndraSnare®


Order Code	Diameter Loop	For Diameter Vessel	Length Loop Wire	Length Catheter	Diameter Catheter
	mm	mm	cm	cm	F
AS-5	5	3 – 5	125	110	4
AS-10	10	6 – 10	125	110	4
AS-15	15	11 – 15	125	110	4
AS-20	20	16 – 20	125	110	4
AS-25	25	21 – 25	125	110	5
AS-25/65	25	21 – 25	65	50	5
AS-30	30	26 – 30	125	110	5
AS-35	35	31 – 35	125	110	5

Order Codes AndraSnare® Micro

Order Code	Diameter Loop	For Diameter Vessel	Length Loop Wire	Length Catheter	Diameter Catheter
	mm	mm	cm	cm	F
ASM-2	2	1 – 2	175	150	2.3 – 3
ASM-4	4	3 – 4	175	150	2.3 – 3
ASM-7	7	5 – 7	175	150	2.3 – 3


Packaging Unit: 1

Loopmaster® - Sochman Snare

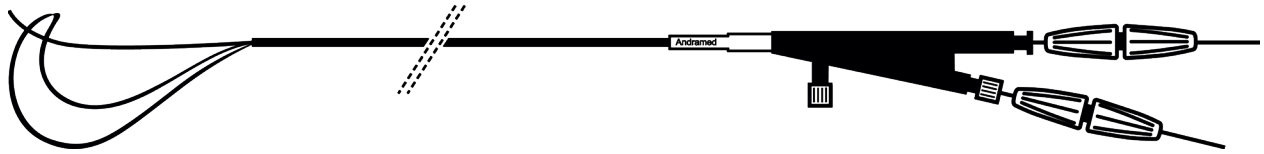


- Nitinol snare for the interventional retrieval of intravascular foreign bodies in the cardiovascular system
- Combination of a snare and a catching wire
 - Flexible nitinol snare that re-assumes its original shape after exiting the catheter
 - Catching wire to fix the foreign body additionally
- Good x-ray visibility through nitinol and additional x-ray marker


Retrieval technique for foreign bodies

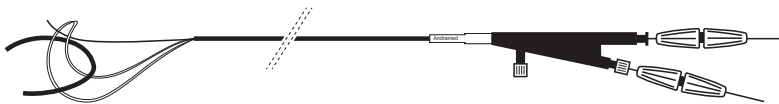

1 Open the snare and place it around the foreign body to be removed.

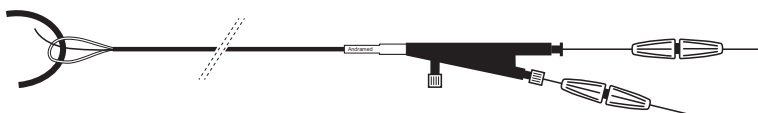

2 By closing the snare, the foreign body is enclosed between the catching wire and the catching snare.

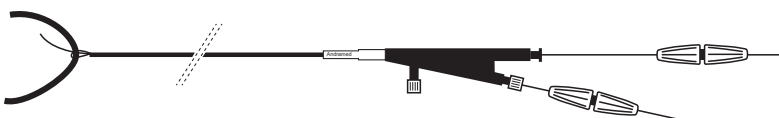

3 This makes it possible to remove the foreign body securely from the body.

Order Codes Loopmaster® - Sochman Snare

Order Code	Diameter Loop	Length Loop Wire/Catching Wire	Length Catheter	Diameter Catheter
	mm	cm	cm	F
ASL-25	25	130/150	120	8

Packaging Unit: 1

Andramed GmbH

Schiesswiesenstr. 18
72766 Reutlingen, Germany
Tel.: +49 (0) 7127 811 57 15
Fax: +49 (0) 7127 811 57 11
info@andramed.com
www.andramed.com



Made in Germany
Rev.06/2024-04-23

